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CERUSTOP® Ohrenöl Spray

Was ist CERUSTOP Ohrenöl Spray?

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CERUSTOP® Ear Oil Spray

What is CERUSTOP Ear Oil Spray?

CERUSTOP Ear Oil Spray is used for removal of excessive or hardened ear wax (cerumen) to prevent inflammation of the external auditory canal.

What does CERUSTOP Ear Oil Spray contain?

Caprylic/Capric Triglyceride, Liquid Paraffin, Isostearyl Isostearate, Almond Oil, PEG-40 Sorbitan Peroleate, BHA/BHT and Vitamin E.

What are the properties of CERUSTOP Ear Oil Spray?

CERUSTOP Ear Oil Spray is sprayed into the external auditory canal. It penetrates into the earwax making it more soluble and therefore easily removable.

Hardened earwax affects the hearing and can cause inflammation of the external auditory canal. It should therefore be removed regularly.

CERUSTOP Ear Oil Spray is also particularly suitable for users of hearing aids.

CERUSTOP Ear Oil Spray can also be used by the physician before rinsing the ear.

Therapeutic Indications – What is CERUSTOP Ear Oil Spray used for?

- > To dissolve and remove excessive or hardened ear wax.
- > To prevent inflammation of the external auditory canal.
- > To clean the auricle and external auditory canal.

CERUSTOP Ear Oil Spray can be used in pregnancy and lactation without hesitation.

CERUSTOP Ear Oil Spray is suitable for adults and children of 3 years and older.

Dosage and Method of administration – How to use CERUSTOP Ear Oil Spray

To remove excessive or hardened earwax it is recommended to spray 2 - 3 puffs (adults) and 1 - 2 puffs (children) of lukewarm CERUSTOP Ear Oil Spray into the external auditory canal. Close the ear with a small piece of cotton wool to prevent the ear oil from dripping out. The cotton wool should be applied loosely and never pressed into the auditory canal! After approximately 10 minutes rinse the ear gently with warm water if possible with a hand held shower.

In cases of extreme hardness it is recommended to repeat the procedure.

Furthermore, to remove earwax, dirt and dead skin attached to the outer ear place a few puffs of lukewarm CERUSTOP Ear Oil on a piece of cotton wool and wipe out the ear several times. Afterwards wipe clean with a dry cloth.

After rinsing with water we recommend the use of NORMISON-Ear Drops to quickly dry and re-establish the natural acid layer of the skin of the external auditory canal.

You should not use CERUSTOP Ear Oil Spray

- > if you are hypersensitive (allergic) to any of the ingredients of CERUSTOP Ear Oil Spray,
- > if the auditory canals are inflamed or when the eardrum is perforated (hole in the eardrum).

Special warnings for safe use

CERUSTOP Ear Oil Spray is not a medicine. In case of earache or extremely hardened plugs of earwax you should seek medical advice! Please talk to your doctor before using CERUSTOP Ear Oil Spray in case your eardrum/middle ear is pre-damaged.

Do not insert any objects (e.g. cotton buds) into the auditory canal!

The bottle should always be kept closed and out of the reach of children!

Storage instructions: Do not store below 8 °C and not above 25 °C.

Pack size: Bottle contains 10 ml with nebulizer. 1 puff = 0.1 ml.

Shelf-life: 36 months. Do not use longer than 8 weeks after first use.

Manufacturer: MoNo chem-pharm Produkte GmbH, Leystraße 129, A-1200 Vienna

Distribution Austria: SIGMAPHARM Arzneimittel GmbH, A-1200 Vienna

Distribution Germany: bene-Arzneimittel GmbH, D-81479 Munich

Revision date: June 2017

CERS-0010-BP-02